


**IMPORTANT! - Please find enclosed a Prusa design 3D Printed Face Shield. Please ensure anyone using the enclosed face shield(s) reads this information first:**

## Assembly instructions

Before starting any assembly **WASH HANDS** for at least 20 Seconds

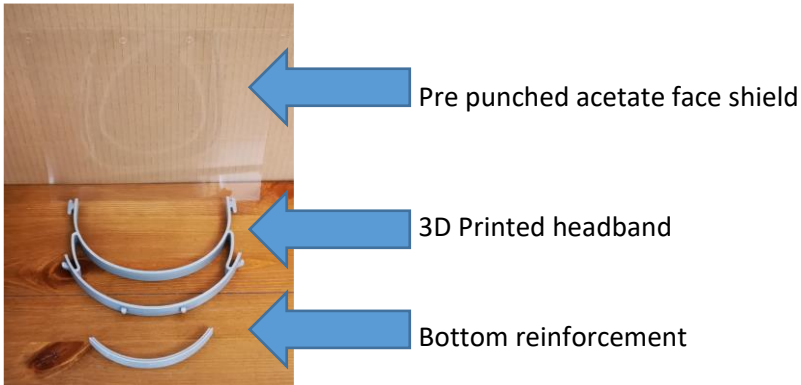

Above: Securing band/strap. Many items can be used including, single use tourniquets, button hole elastic or even large elastic bands



Step 1. Line up the first hole of the acetate sheet with the first peg on the head band and push the acetate onto the peg. The hole in the acetate is slightly smaller than the peg so it stretches and grips the peg firmly.

Step 2. Continue working around the headband pushing the acetate on to each peg.

Step 3. When you reach the last peg, pull the acetate tight and push the last hole over on the acetate over the last peg.

Step 4. The visor should now be secured to the head band. It can easily be removed and replaced if required.



Step 5. Push the bottom reinforcement piece onto the bottom of the acetate face shield, it should grip nicely. Tip - sometimes it's easier to slide it on from one end.)



Step 6. You should now have something that looks like this.



Step 7. Push the securing strap over the T-piece on each side of the headband.



Step 8. You should now have a full assembled face shield. Stay safe, and thank you for all you are doing to keep everyone safe and well during this crisis.

If you need more copies of these instructions a digital file can be found at:

<http://www.pccareuk.com/covid19ppe/> in the **FOR RECIPIENTS** section or use the QR code or the right >

